



The new **PLUS** for wet therapy:  
TenderWet® plus.

The innovative wound dressing pad from HARTMANN.



Wound Management

# The new **PLUS** in quality.

## Up to 3 days without changing dressings.




### The new wound dressing pad for wet therapy:

- **Economical:** The efficient 72-hour solution saves time and expenses.
- **Practical:** Dressing changes without adherence.
- **Reliable:** Germs are killed by polyhexanide in the wound dressing pad.
- **Versatile:** TenderWet plus for superficial wounds and TenderWet plus cavity for deep wounds.



### The new wound dressing pad that convinces users:

In an observational study carried out with 170 patients, three consecutive dressing changes were documented in each case:

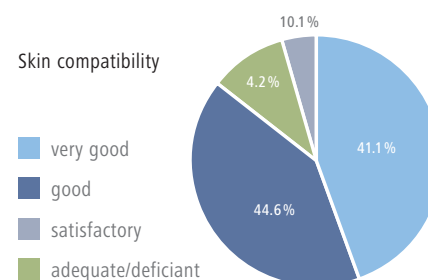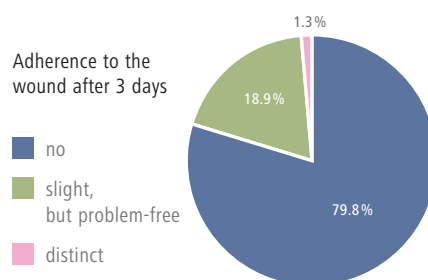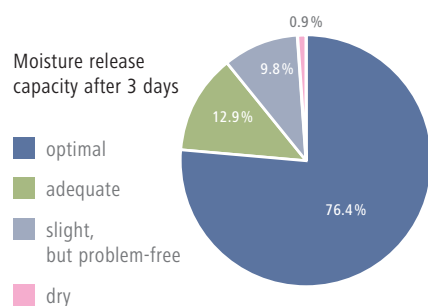

**The result: Maximum satisfaction.** TenderWet plus meets strict user requirements.

### The new wound dressing pad for 72 hours:

- **No drying out:** The special material and special design effectively counteract drying out.



### The new wound dressing pad for more safety:

- **No contamination:** The antiseptic polyhexanide kills germs in the wound dressing recontamination and prevents recontamination of the wound.
- **No pain:** The non-stick silicone coating strips minimise the risk of adherence to the wound.






# The new **PLUS** in know-how. For effective wet therapy.


## Absorbing and rinsing mechanism:

- **Absorption:** TenderWet plus absorbs necroses, cellular components, bacteria and exudate.
- **Rinsing effect:** At the same time, Ringer's solution is continuously released.
- **Effect:** The wound is cleansed and kept moist.

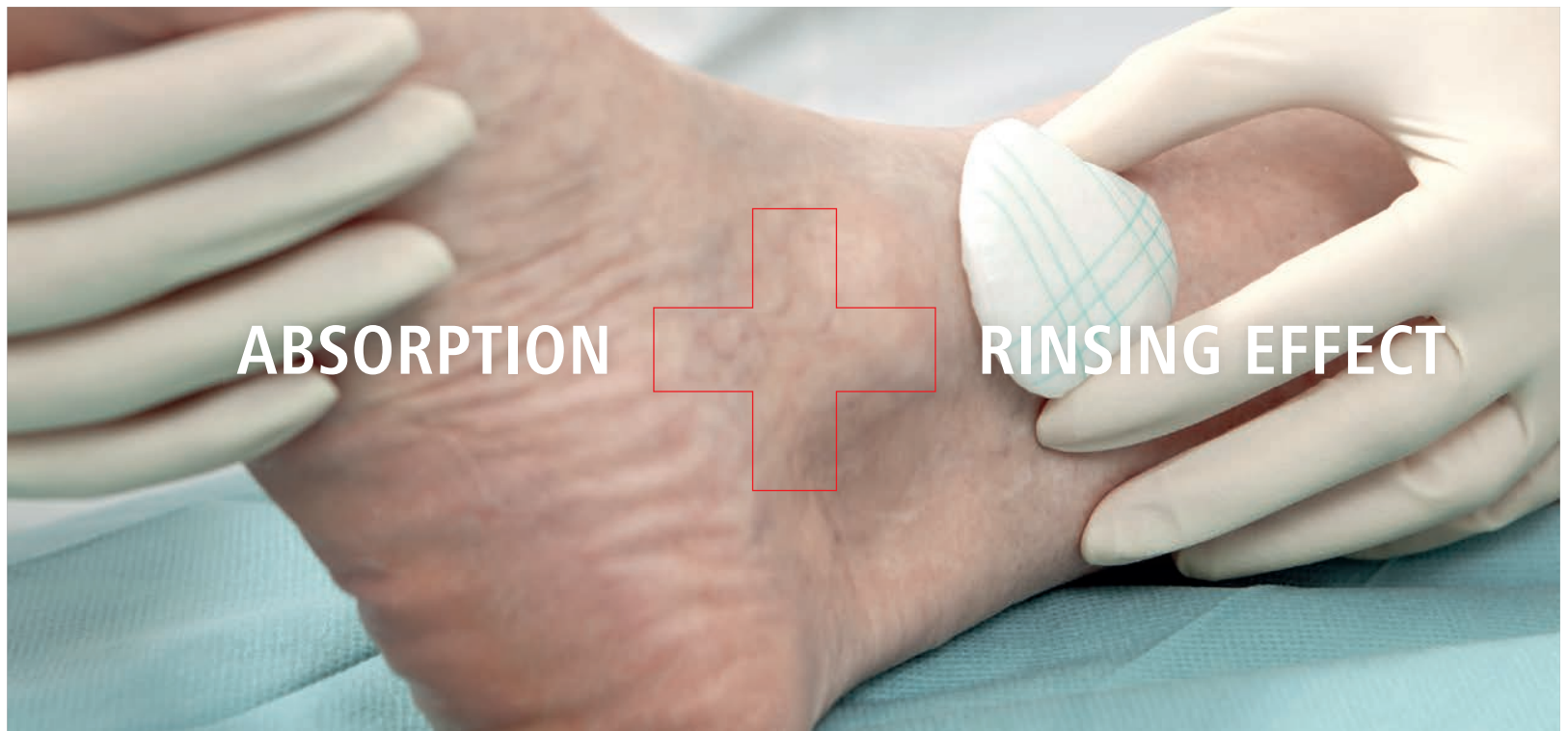


## The stimulus for healing is set:


The tried and tested absorbing and rinsing mechanism of TenderWet plus guarantees outstanding wound cleansing.

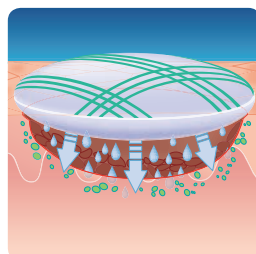

Release of Ringer's solution

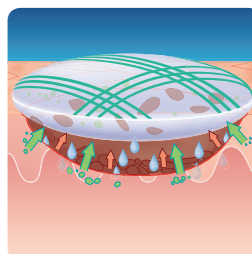

Absorption of necroses,  
cellular components,  
bacteria and exudate


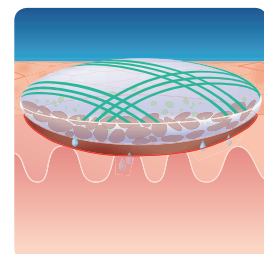


Saturated wound dressing pad


# The new **PLUS** in treatment success. For all types of wounds.

## Positive effect on wound healing:

- **Regulation:** The excessive activity of healing-inhibiting matrix metalloproteinases (MMP) is significantly reduced.
- **Inhibition:** The relatively low pH of the wound dressing pad inhibits bacterial growth.
- **Healing:** Since these healing-inhibiting factors are eliminated, even chronic wounds can heal again.

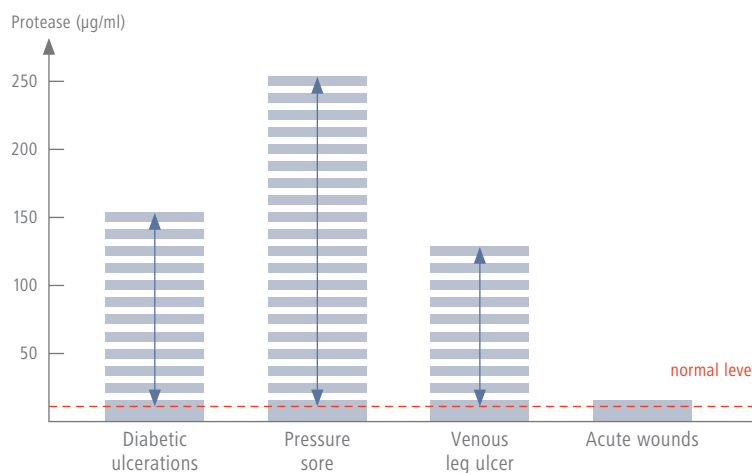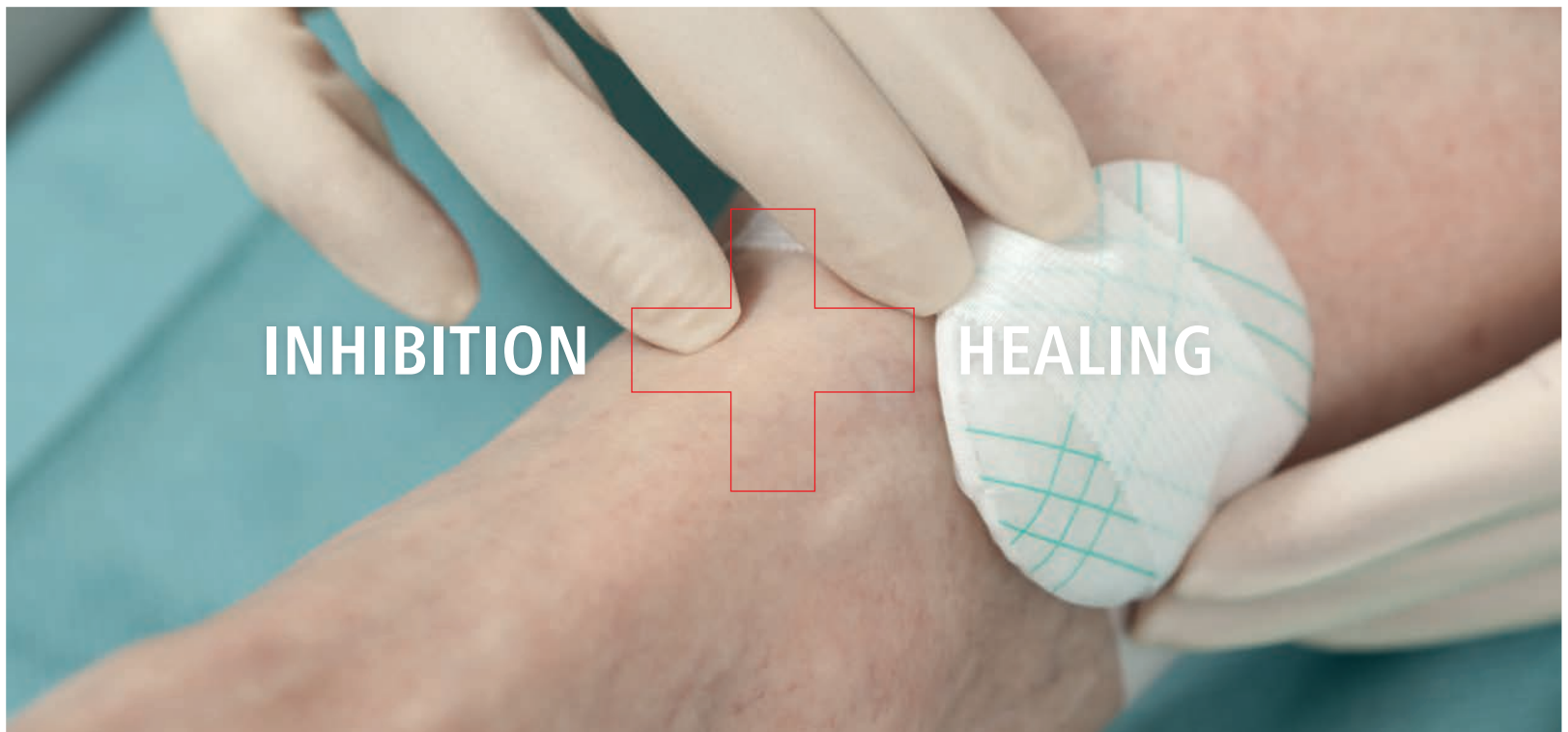

MMP concentration ranges can vary depending on the type of wound.  
Source: Trengove N.J. et al., Wound Repair and Regeneration 1999; No. 7: 442-452.

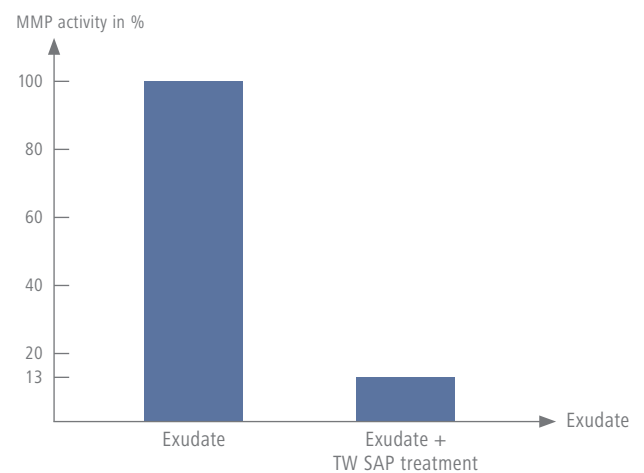


The polyacrylate super absorber (SAP) that is in the wound dressing pad reduces the MMP-2 and MMP-9 activity in the exudate of chronic wounds.

# The new **PLUS** in variety.

## TenderWet® plus and TenderWet® plus cavity.

Ideal for superficial and deep wounds:

- **TenderWet plus for superficial wounds:** The knitted polypropylene fabric and the thin silicone strips on the side facing towards the wound prevent adherence to the wound.
- **TenderWet plus cavity for deep wounds:** Can be easily packed and used in combination with TenderWet plus.

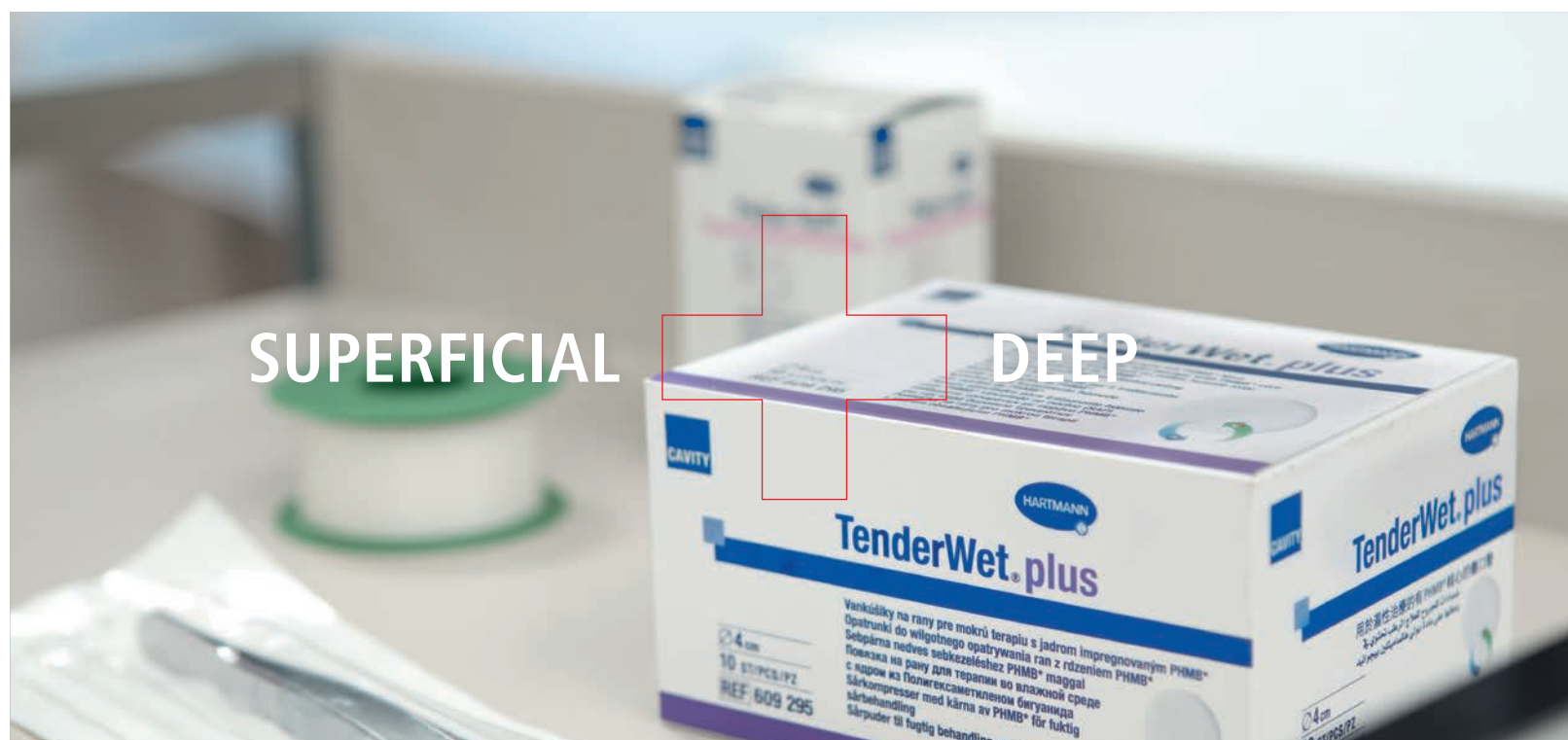

Ideal for all wounds:

- Acute and chronic wounds.
- Especially in the cleansing phase.
- Infected as well as non-infected wounds.

# The new **PLUS** in innovation.


## Well-thought-out products for wet therapy.

### Successful healing instead of drying out:

- The mixture of SAP and cellulose fibres (air-laid) keeps the wound moist.
- Ringer's solution is continuously released; necroses and fibrin are loosened and absorbed.
- A particular, special welded covering film largely prevents premature drying out of the wound dressing pad and thereby of the wound too.

### Comfort instead of adherence:

- The moist knitted polypropylene fabric does not tend to stick.
- The silicone strips also counteract adherence.

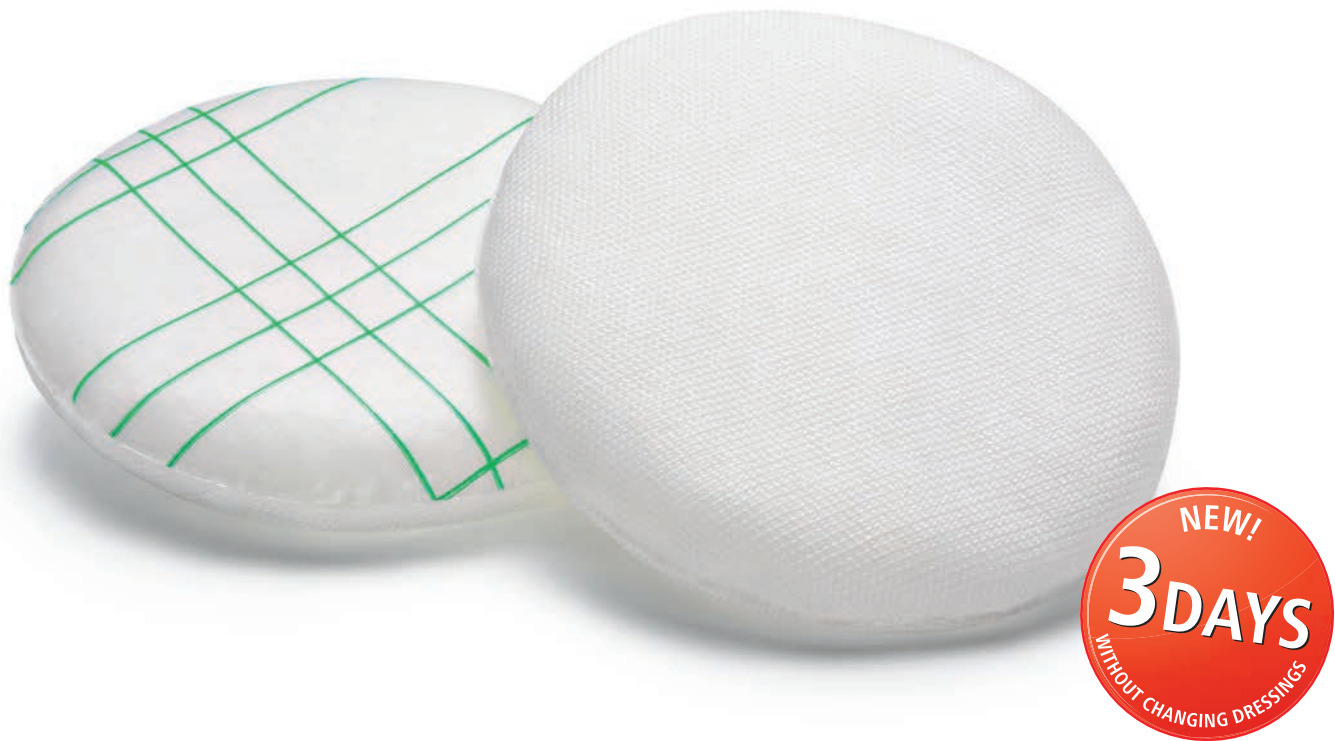

### Effectiveness instead of contamination:

- The polyhexanide prevents germ load in the pad and thereby protects the wound against recontamination.

### Economic efficiency instead of dressing changes:

- Its use saves time and money.
- Up to 3 days without changing dressings.



helps healing.

PAUL HARTMANN AG  
D-89522 Heidenheim  
Germany

Visit our website:  
[www.hartmann.info](http://www.hartmann.info)



Presentation	Size	Ref. No.	Unit	Units per Case
TenderWet® plus	4 cm round	609 291/0	Box of 10	6
	5.5 cm round	609 292/0	Box of 10	6
	7.5 x 7.5 cm	609 293/0	Box of 10	6
	10 x 10 cm	609 294/0	Box of 10	6
TenderWet® plus cavity	4 cm round	609 295/0	Box of 10	6
	7.5 x 7.5 cm	609 296/0	Box of 10	6

(02/12)